

FemCap

Reusable, hormone-free
contraception




- More natural and easy to use
- Reusable up to a year
- Continuous protection for 48 hours
- Safe, effective and latex-free



www.femcapcanada.ca

How the FemCap Works

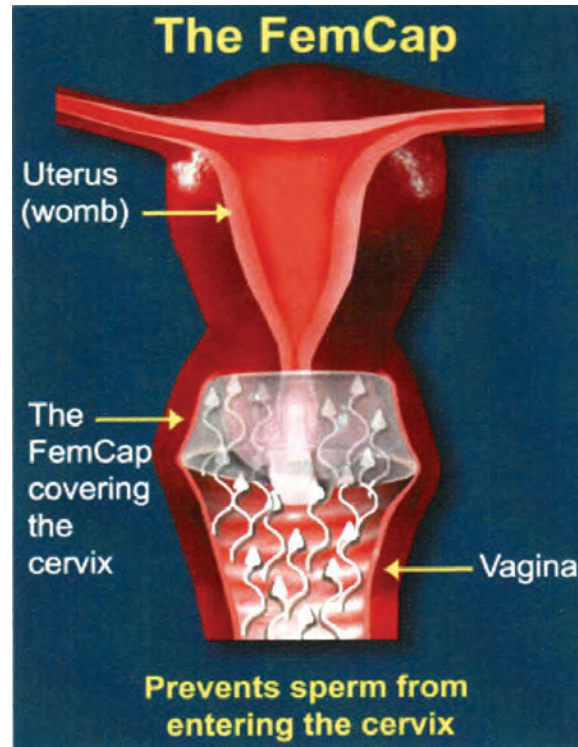


After placing the FemCap, follow the instructions included with the product — the FemCap will protect you by preventing sperm from entering the cervix.

Advantages of the FemCap

- Latex-free barrier contraceptive device
- Easy to use, effective, hormone-free
- Proven in clinical trials to be over 92% effective
- Remains in place for multiple uses within a 48 hours period
- Designed to cover the cervix to prevent sperm from entering
- Made of soft, medical grade silicone
- Safe, no side effects
- Reusable for one year
- Does not interfere with the sexual desire or the pleasure of either partner
- Three sizes: 22mm, 26mm and 30mm

FemCap is available in three sizes:

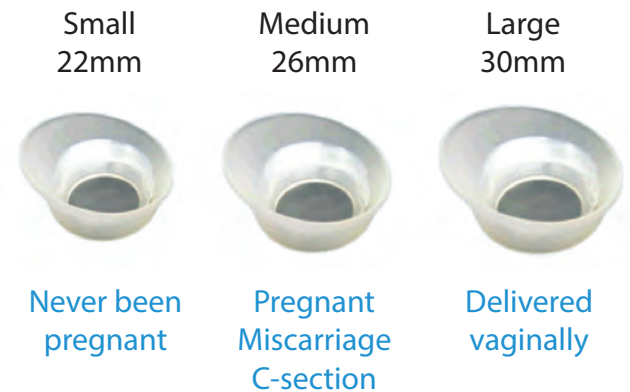

How to use the FemCap

The FemCap can be applied from 15 minutes up to 40 hours before intercourse.

1. Put spermicide in both sides of the FemCap.



2. Fold the FemCap and insert it in your vagina.



3. Push the FemCap as far as possible to place the large border behind the cervix.



4. After the intercourse, wait at least 6 hours before removing the FemCap. To remove it, grab the handle and gently pull out.



Detailed instructions are included with every FemCap.

Instructional video at:

www.femcapcanada.ca/femcap--use

Frequently Asked Questions

Can the FemCap get dislodged during sex?

No. The FemCap is specifically designed to prevent it from being dislodged during intercourse. To accommodate different sexual positions, the FemCap is designed to rotate.

How to clean the FemCap?

The FemCap should be washed with anti-bacterial hand soap and rinsed with tap water after each use.

When to replace the FemCap?

It is recommended that the FemCap be replaced every year.

Can the FemCap be used during menstruation?

No, as it would impede the natural flow of menstrual fluid.

What about spermicide irritation?

It is true that spermicide can cause irritation when used very frequently and in very high doses. To minimize it, the FemCap is designed with a unique groove which stores the bulk of the spermicide and minimizes irritation.



Order FemCap Now !

Online: www.femcapcanada.ca
or call 1.855.246.2442



FemCapCanada.ca

FemCap Canada division of
MEDISAFE DISTRIBUTION INC.

7 – 11771 Horseshoe Way
Richmond, BC V7A 4V4 Canada

www.femcapcanada.ca

