REUSABLE, HORMONE-FREE CONTRACEPTION

Advantages

- Latex-free barrier contraceptive device that prevents sperm from entering the cervix
- Can be applied from 15 minutes up to 40 hours before intercourse
- Easy to use, effective, hormone-free
- Proven in clinical trials to be over 92% effective
- Remains in place for multiple uses within a 48 hour period
- Made of soft, medical grade silicone
- Safe, no side effects
- Reusable for one year
- Does not interfere with the sexual desire or the pleasure of either partner

How to use the FemCap

1. Put spermicide in both sides of the FemCap.


2. Fold the FemCap and insert it in your vagina.

3. Push the FemCap as far as possible to place the large border behind the cervix.

4. After intercourse, wait at least 6 hours before removing the FemCap. To remove it, grab the handle and gently pull out.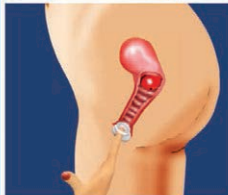

| Small 22mm | Medium 26mm | Large 30mm |  |
|--|---|--|---|
|  Never been pregnant |  Pregnant, Miscarriage, C-section |  Delivered vaginally | |

Three sizes

Frequently Asked Questions

Can the FemCap get dislodged during sex?

No. The FemCap is specifically designed to prevent it from being dislodged during intercourse. To accommodate different sexual positions, the FemCap is designed to rotate.

How to clean the FemCap?

The FemCap should be washed with anti-bacterial hand soap and rinsed with tap water after each use.

When to replace the FemCap?

It is recommended that the FemCap be replaced every year.

Can the FemCap be used during menstruation?

No, as it would impede the natural flow of menstrual fluid.

What about spermicide irritation?

It is true that spermicide can cause irritation when used very frequently and in very high doses; to minimize this, the FemCap is designed with a unique groove which stores the bulk of the spermicide in it and this helps to minimize the possibility of irritation.

FemCap Canada division of

MEDISAFE DISTRIBUTION INC.

7 – 11771 Horseshoe Way, Richmond, BC V7A 4V4 Canada

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