FemmyCycle

INNOVATIVE MENSTRUAL CUP

*New alternative to pads
and tampons*

Advantages

- Innovative menstrual cup
- Ideal for active, athletic women
- Comfortable and discreet as a tampon
- Keeps menstrual flow from spilling
- Collects menstrual flow for up to 12 hrs
- Easy to insert and remove
- Made of soft medical grade silicone
- Available in three sizes
- Not to be used if you have ever had Toxic Shock Syndrome (TSS).

Three sizes

How to use the FemmyCycle

How to insert the FemmyCycle®

1 Sit on the toilet with your knees apart, or squat or stand with one leg raised on the toilet seat.

2 Close the lid of the FemmyCycle® and compress the Rim **firmly**, as shown in the figure.

3 Fold the rim at the notch into a tight "U" shape while gripping it **firmly**, trapping the air inside the receptacle. (The trapped air will help it to pop open when it is inserted inside the vagina.)

4 Hold the folded FemmyCycle® rim **firmly** with index finger and thumb, separate your labia with other hand and insert the FemmyCycle® slowly. This will help it to pop open.

5 Push the FemmyCycle® backward. **Stop pushing** once the removal ring enters your vagina.
NOTE: The FemmyCycle® is designed not to open completely. This will create suction to draw the flow into the device.

6 Once the FemmyCycle® is inserted into the vagina, the rim will open enough to collect your menstrual flow.

NOTE: The FemmyCycle® should sit in the lower part of the vagina as a receptacle for menstrual blood.

Frequently Asked Questions

Is the FemmyCycle easy to clean?

Yes. Just empty the content into the toilet and clean the FemmyCycle with liquid soap and warm water, then rinse thoroughly.

Will you experience any leakage using the FemmyCycle overnight?

Unlike tampons, the FemmyCycle is quite safe and effective for overnight use.

Will I feel the FemmyCycle while using it?

No. When inserted properly, the FemmyCycle is as comfortable and discreet as a tampon.

Can I use the FemmyCycle if I am also using an IUD?

Yes. It's safe and effective for use with an IUD.

Can I have intercourse while wearing the FemmyCycle?

Not penetrative intercourse, however you may enjoy cleaner non-penetrative intercourse when using the FemmyCycle.

FemmyCycle.ca division of

MEDISAFE DISTRIBUTION INC.

7 – 11771 Horseshoe Way, Richmond, BC V7A 4V4 Canada

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